PERLUX® UV-370

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INSTRUCTIONS

ESOGETICS
heilkraft der farben

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Beispiele:

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1. Purpose and Effect

The treatment with the Perlux system is an electromagnetic irradiation which is applied on points or small areas on the skin in a kind of light with constant frequency.

As very low voltages are used, an infinitesimal energy transfer takes place. The effect is based on the resonance of the applied light with a constant frequency.

In doing so, the sensitive sensors of the surface of the skin and their interdependency and communication with the command organs of the body are stimulated.

The Perlux colorpunctur devices are used in the following medical fields:

Sporting injuries (contusions, strains, hardenings), complaints of bones and joints, irritations of muscles, nervous pains, pains in the musculoskeletal system, as well as consequences of traumas and chronic degenerative diseases.

Treatments on the surface of the skin are possible as well as treatments in a kind of acupuncture are possible with the Perlux color therapy system.

2. Startup:

The Perlux UV-370 is ready for use, following the insertion of 4 x 1.5 volt batteries of the LR1 type. Open by turning the cap on the flat end of the pen, and insert the batteries with positive polarity pointing inward.

Please to not use any batteries not of this type designation.

Chargeable batteries may not be used.

The period of service with a new set of batteries is circa 8-10 weeks.

3. Operating the Perlux UV-370:

In order to activate the device, press the black button on the head piece of the device once.

- The red indicator lamps show that the device is ready for operation.
- The diode at the point of the device is strongly illuminated with light-violet.
- To switch off, you also press the black button once. The diode and indicator lamps should no longer be illuminated.
- **Do not exert pressure on the diode.**

Point locator:

- The wide end of the pen can be used for stimulation and massage.
- The pointed end of the pen serves the purpose of localizing points for treatment.

The application of the light on certain points and zones upon the skin as well as the duration of the irradiation is always determined by the therapist according to the therapeutical concept.

Additional references:

Peter Mandel: „Practical Compendium of Colorpuncture (Part 1 and 2)“, „The Ophthalmotropic Genetic Therapy (OGT)“, „Sense and Nonsense of Sickness and Pain“

Dr. Fausto N. Pagnamenta: „Children in the Light“

4. Cleaning:

To clean the surface of the device, as well as the diode at the point, you may use all mild cleaning agents and disinfectants. For this purpose, only use a dampened cloth.

Attention: Liquids must not be allowed to enter the interior of the Perlux UV-370.

Never attempt to sterilize the device!

Organic solvents and strong household cleaning agents must never be used to clean the device. In the case of longer periods of non-use (longer than 5 days), please remove the batteries, otherwise the device may be damaged (battery leakage).

5. Contraindications, Precautions and Side Effects:

No side effects are known to us.

Important: Never beam into open or closed eyes with the device Perlux UV-370!

The device has been exclusively designed for therapists and may only be used by trained personnel.

This ultraviolet therapy is based upon the somatotopies, zones and points, which have been developed over the many years that Mr. Peter Mandel has worked in his practice. The effects on zones and points, which may have been derived from other UV therapies, have not been checked.

For this reason, we explicitly recommend **not using unchecked applications**, and we assume no liability for the effects on patients.

6. Inspection, Maintenance and Repairs

The Perlux UV-370 does not require regular maintenance. In case of uncertainties or complaints about the functionality, please contact the manufacturer directly: esogetics GmbH in Bruchsal, Germany.

Only the manufacturer or persons appointed by the manufacturer are authorized to repair or to make other changes to the instrument. We assume no liability for inappropriate changes and handling that does not conform to the described purpose of use.

The warranty period for the instrument is one year from the purchase date.

Please contact the manufacturer directly, if your instrument does not work properly although you have handled it correctly. The manufacturer will immediately repair the device or replace the de-

fective one with an equivalent device.

7. Specifications

Device type:	Light radiation (clinical function, no essential performance)
Type designation:	Perlux UV-370
Power supply:	6 V/4 x 1.5 V LR1 batteries
Nominal current:	10 mA
Type of construction:	hand-held device
Protection rating:	III
Output frequency:	370 nm
Weight:	90 g with batteries
Dimensions:	L:17cm ø:20mm
Operating conditions:	5° C to 35° C
Storage:	-20° C to 70° C
Air pressure:	800 hPa to 1030 hPa
Humidity:	30% to 93%
Spectral irradiance or spectral radiant exposure for all intended configurations of UV-370	See page 13
Maximum output of optical radiation for all intended configurations of UV-370	See page 13

Contents:

- 1 Light radiation device UV-370
- 1 Point locator
- 4 Batteries of the LR1 type
- 1 Operating manual

Applied Standards and Guidelines

Electrical safety: IEC 60601-1:2005+AM1:2012 (hand-held, internally powered, no applied parts)

Electromagnetic compatibility: IEC 60601-1-2:2014 (group 1, class B; professional healthcare environment)

Photobiological safety of lamps and lamp systems (IEC 62471:2006)

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<http://www.esogetics.com>

Info-de@esogetics.com

esogetics is a registered medical device manufacturer. We adhere to a certified and monitored quality management system according to Medical Device Directive 93/42/EEG and to DIN EN ISO 14385:2012 with an integrated monitoring and reporting system.

Appendix:

The conformity with the guidelines of relevance to the device is confirmed by the CE symbol with the number of the notified body.

Product:

The crossed-out refuse bin symbol on your product indicates that it is an electric and electronic device, for which special conditions of disposal apply. For the implementation of recycling, for WEEE disposal in accordance with the directive governing “Waste Electrical and Electronic Equipment”, and for the protection of the environment and health, European directives require the selective collection of equipment in one of the two following manners:

- Your dealer willing accepts your old equipment, when you purchase new equipment from him.
- Equipment can also be disposed of at collecting points intended for this purpose.

Instructions / follow manual

Batteries:

Old batteries must be disposed of at collecting points intended for this purpose.

Serial-No.

The Serial-No. of your Perlux device is built as follows:

XX **YY** - **XXXX** — Serial number.

└ Year of Production

└ Designation of the device

(**CP** = P-117; **CF** = F-333; **UV** = UV-370; **IR** = IR-950)

Examples:

IR18-0145 = Perlux IR-950 from the year 2018 with the no. 145.

UV17-0937 = Perlux UV-370 from the year 2017 with the no. 937.

Emission limits for risk groups of continuous wave lamps – UV-370					
Risk	Action spectrum	Symbol	Units	Exempt	
				Limit	Result
Actinic UV	$S_{UV}(\lambda)$	E_S	$W \cdot m^{-2}$	0.001	3.8E-05
Near UV		E_{UVA}	$W \cdot m^{-2}$	10	0.378
Blue light	$B(\lambda)$	L_B	$W \cdot m^{-2} \cdot sr^{-1}$	100	0.474
Blue light, small source	$B(\lambda)$	E_B	$W \cdot m^{-2}$	1.0	N/A
Retinal thermal	$R(\lambda)$	L_R	$W \cdot m^{-2} \cdot sr^{-1}$	$28000/\alpha$	143
Retinal thermal weak visual stimulus	$R(\lambda)$	L_{IR}	$W \cdot m^{-2} \cdot sr^{-1}$	$6000/\alpha$	78
IR radiation eye		E_{IR}	$W \cdot m^{-2}$	100	— ¹⁾

¹⁾ value obtained below the detection limit of the measuring equipment

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